JetCare Med by TavTech

Tretman obogaćen osmolitima za
revitalizaciju kolagena i elastina | Unapredite
rezultate svoje ordinacije

Formulisano za lekare, kreirano za rezultate.

JetCare Med by TavTech je ekskluzivna kolekcija JetPeel infuzionih preparata obogaćenih osmolitima, osmišljena za dermatologe i estetske stručnjake kako bi svojim pacijentima ponudili napredan JetPeel tretman za stimulaciju kolagena i elastina, sa klinički dokazanim rezultatima.

Liniju JetCare Med čine četiri infuzije i jedan „booster“ tretman:

- **JetCare Med Refill Infusion** – popunjavanje i zaglađivanje kože
- **JetCare Med Relight Infusion** – ujednačavanje tena i posvetljivanje
- **JetCare Med Relax Infusion** – opuštanje mišića lica
- **JetCare Med Revitalize Infusion** – anti-age delovanje
- **JetCare Med Refine Boost Infusion** – završni efekat podizanja i sjaja

Svi proizvodi iz JetCare Med linije formulisani su sa jedinstvenom i bogatom koncentracijom osmolit – organskih molekula male molekulske mase, koji deluju kao efikasni nosači za zaštitu peptida i očuvanje stabilnosti proteina, doprinoseći zdravijem i revitalizovanom kolagenu i elastinu.

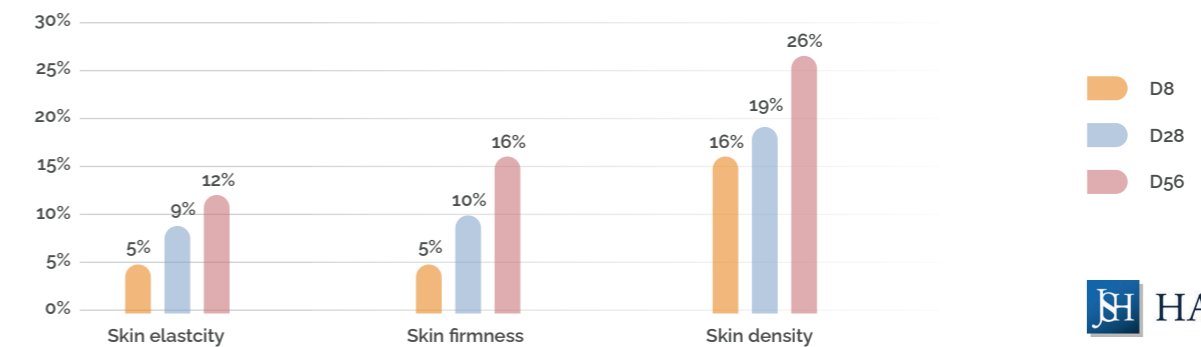
Infuzije se primenjuju pomoću MDJet by TavTech nastavka, specijalno dizajniranog za JetPeel tretmane sa JetCare Med serumima. Preparati su upakovani u špriceve od 5 ml, koji se lako i brzo povezuju sa MDJet nastavkom, obezbeđujući praktičan i neprekinut tok tretmana.

JetCare Med: Rezultati govore sami za sebe

Klinička studija* sprovedena sa JetCare Med i JetPeel tretmanima pokazala je izuzetna poboljšanja gustine, čvrstine i elastičnosti kože.

Osam nedelja nakon prvog od ukupno tri tretmana, izvedenih u periodu od osam dana, ispitanici su pokazali**:

- **26% poboljšanja gustine kože**
- **16% poboljšanja čvrstine kože**
- **12% poboljšanja elastičnosti kože**

 HAMILTON

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
Skin Density Improvements

Measured with Dub® Skin Scanner manufactured by TPM Taberna Pro Medicum GmbH

JS HAMILTON REPORT No. 416345-1/24/GDA/PK | JETCARE MED TREATMENT

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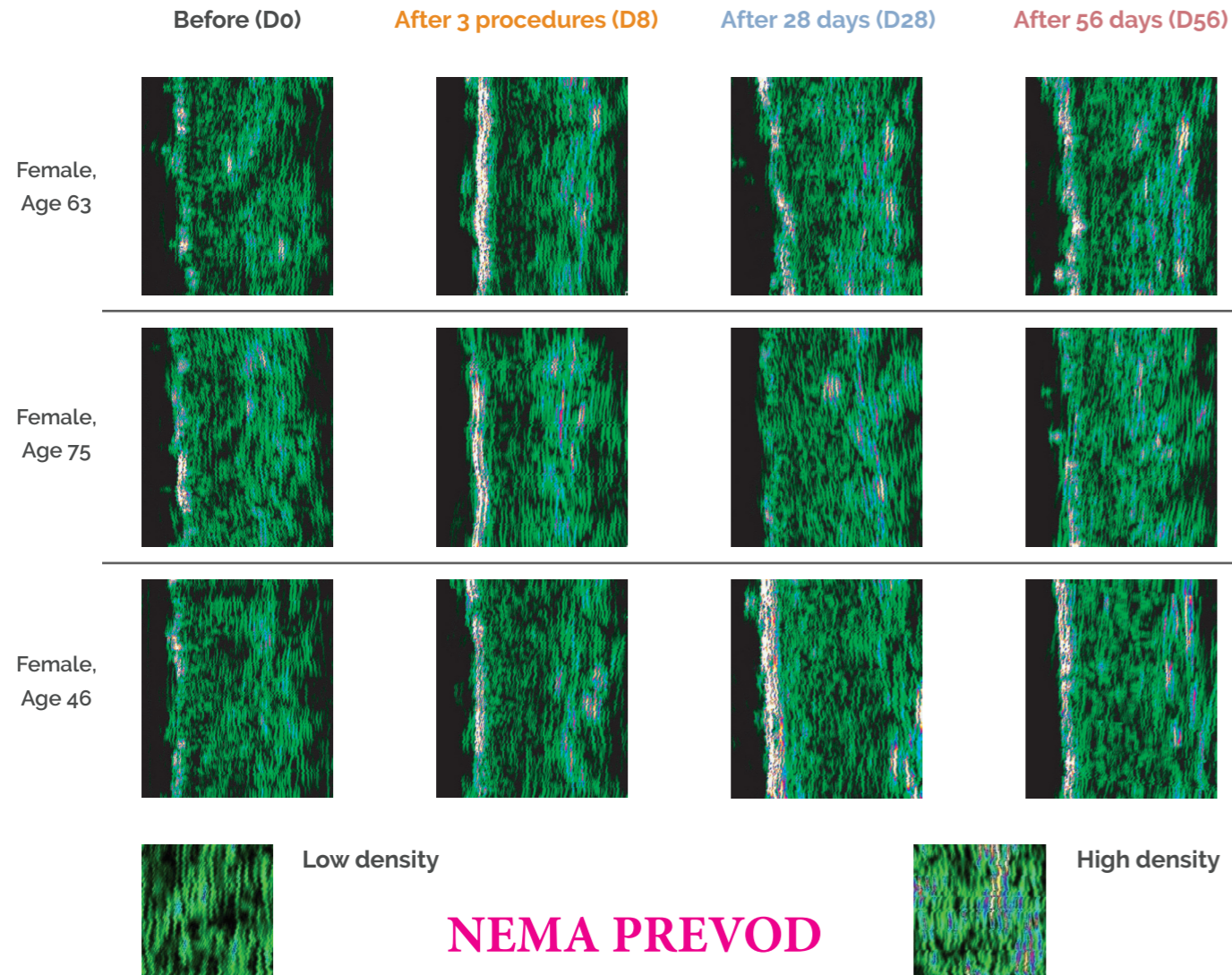Re-ignite your clinic results with osmolyte-enriched collagen and elastin energizers for clinically proven improved skin elasticity, skin firmness and skin thickness

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Argireline® Amplified Peptide Solution C – 5%

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Argireline® Amplified Peptide Solution C – 10%

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JetCare Med Treatment Recommendations and Guidelines

Anti-Aging Treatment for all Skin Types

	LYMPHATIC MASSAGE	STEP 01	EXFOLIATION	STEP 02	INFUSION	STEP 03	BOOST INFUSION	STEP 04
 Handpiece	TripleJet		TripleJet		MDJet		MDJet	
 Distance from skin	4-7 cm		1 cm		0.5-0.8 cm		0.5-0.8 cm	
 Handpiece Angle	90°		90°		90°		90°	
 Time	10-15 minutes		5-7 minutes		5-7 minutes		5-7 minutes	
1st Treatment	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 16%		5ml syringe of JetCare Med Relax		5ml syringe of JetCare Med Refine Boost Infusion	
2nd Treatment 7-10 days after 1st Treatment	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 10%		5ml syringe of JetCare Med Relight		5ml syringe of JetCare Med Refine Boost Infusion	
3rd Treatment 7-10 days after 2nd Treatment	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 16%		5ml syringe of JetCare Med Refill		5ml syringe of JetCare Med Refine Boost Infusion	
4th Treatment 7-10 days after 3rd Treatment	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 10%		5ml syringe of JetCare Med Revitalize		5ml syringe of JetCare Med Refine Boost Infusion	
5th Treatment 7-10 days after 4th Treatment	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 16%		5ml syringe of JetCare Med Relax		5ml syringe of JetCare Med Refine Boost Infusion	
Maintenance Every 6-8 weeks	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 16%		5ml syringe of JetCare Med Relax		5ml syringe of JetCare Med Refine Boost Infusion	
 Remarks			Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro. TripleJet handpiece at 45° - 60°					

JetCare Med Treatment Recommendations and Guidelines

Loss of Tone and Wrinkles Treatment for all Skin Types

	LYMPHATIC MASSAGE	STEP 01	EXFOLIATION	STEP 02	INFUSION	STEP 03	BOOST INFUSION	STEP 04
 Handpiece	TripleJet		TripleJet		MDJet		MDJet	
 Distance from skin	4-7 cm		1 cm		0.5-0.8 cm		0.5-0.8 cm	
 Handpiece Angle	90°		90°		90°		90°	
 Time	10-15 minutes		5-7 minutes		5-7 minutes		5-7 minutes	
1st Treatment	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 16%		5ml syringe of JetCare Med Relax		5ml syringe of JetCare Med Refine Boost Infusion	
2nd Treatment 7-10 days after 1st Treatment	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 10%		5ml syringe of JetCare Med Refill		5ml syringe of JetCare Med Refine Boost Infusion	
3rd Treatment 7-10 days after 2nd Treatment	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 16%		5ml syringe of JetCare Med Relight		5ml syringe of JetCare Med Refine Boost Infusion	
4th Treatment 7-10 days after 3rd Treatment	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 10%		5ml syringe of JetCare Med Revitalize		5ml syringe of JetCare Med Refine Boost Infusion	
5th Treatment 7-10 days after 4th Treatment	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 16%		5ml syringe of JetCare Med Relax		5ml syringe of JetCare Med Refine Boost Infusion	
Maintenance Every 6-8 weeks	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 16%		5ml syringe of JetCare Med Relax		5ml syringe of JetCare Med Refine Boost Infusion	
 Remarks			Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro. TripleJet handpiece at 45° - 60°					

JetCare Med Treatment Recommendations and Guidelines

Post-Acne Treatments for all Skin Types

	LYMPHATIC MASSAGE	STEP 01	EXFOLIATION	STEP 02	INFUSION	STEP 03	BOOST INFUSION	STEP 04
 Handpiece	TripleJet		TripleJet		MDJet		MDJet	
 Distance from skin	4-7 cm		1 cm		0.5-0.8 cm		0.5-0.8 cm	
 Handpiece Angle	90°		90°		90°		90°	
 Time	10-15 minutes		5-7 minutes		5-7 minutes		5-7 minutes	
1st Treatment	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 16%		5ml syringe of JetCare Med Refill		5ml syringe of JetCare Med Refine Boost Infusion	
2nd Treatment 7-10 days after 1st Treatment	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 10%		5ml syringe of JetCare Med Relax		5ml syringe of JetCare Med Refine Boost Infusion	
3rd Treatment 7-10 days after 2nd Treatment	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 16%		5ml syringe of JetCare Med Relight		5ml syringe of JetCare Med Refine Boost Infusion	
4th Treatment 7-10 days after 3rd Treatment	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 10%		5ml syringe of JetCare Med Revitalize		5ml syringe of JetCare Med Refine Boost Infusion	
5th Treatment 7-10 days after 4th Treatment	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 16%		5ml syringe of JetCare Med Relight		5ml syringe of JetCare Med Refine Boost Infusion	
Maintenance Every 6-8 weeks	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 16%		5ml syringe of JetCare Med Relax		5ml syringe of JetCare Med Refine Boost Infusion	
 Remarks			Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro. TripleJet handpiece at 45° - 60°					

JetCare Med Treatment Recommendations and Guidelines

Lightening and Pigmentation Treatments for all Skin Types

	LYMPHATIC MASSAGE	STEP 01	EXFOLIATION	STEP 02	INFUSION	STEP 03	BOOST INFUSION	STEP 04
 Handpiece	TripleJet		TripleJet		MDJet		MDJet	
 Distance from skin	4-7 cm		1 cm		0.5-0.8 cm		0.5-0.8 cm	
 Handpiece Angle	90°		90°		90°		90°	
 Time	10-15 minutes		5-7 minutes		5-7 minutes		5-7 minutes	
1st Treatment	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 16%		5ml syringe of JetCare Med Relight		5ml syringe of JetCare Med Refine Boost Infusion	
2nd Treatment 7-10 days after 1st Treatment	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 10%		5ml syringe of JetCare Med Relax		5ml syringe of JetCare Med Refine Boost Infusion	
3rd Treatment 7-10 days after 2nd Treatment	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 16%		5ml syringe of JetCare Med Refill		5ml syringe of JetCare Med Refine Boost Infusion	
4th Treatment 7-10 days after 3rd Treatment	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 10%		5ml syringe of JetCare Med Revitalize		5ml syringe of JetCare Med Refine Boost Infusion	
5th Treatment 7-10 days after 4th Treatment	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 16%		5ml syringe of JetCare Med Relight		5ml syringe of JetCare Med Refine Boost Infusion	
Maintenance Every 6-8 weeks	~18ml of JetCare Hydro		~5ml of Renewal Care Glycolic Acid 16%		5ml syringe of JetCare Med Relight		5ml syringe of JetCare Med Refine Boost Infusion	
 Remarks			Leave on skin for 5-8 min. then wash with ~8ml of JetCare Hydro. TripleJet handpiece at 45° - 60°					

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 [jetpeeltavtechserbia](https://www.linkedin.com/company/jetpeeltavtechserbia)

 **JETPEEL**
by TavTech
Serbia

JETPEEL HANDPIECE
MDJet
Jet Pressure Energy
NOT STERILE

JetCare Med
RELIGHT
Osmolyte enriched
TREATMENTS
Relight Infusion
Refine Boost Infusion
5 syringes
5 syringes

JCM-RELIGHT BOX
5x1.0 cc • 5 syringes (1.0 cc) each
12.5 cc • 5 syringes (1.0 cc) each

 **JETCARE**
by TavTech